


Easy on-PC | SystemOne Quick User Guide




This instruction guide is intended for users that have the EasyOne Connect software integrated with tpp SystemOne patient management system. This simple 8 step guide shows how to perform a test using both systems from importing patient data to exporting test results.

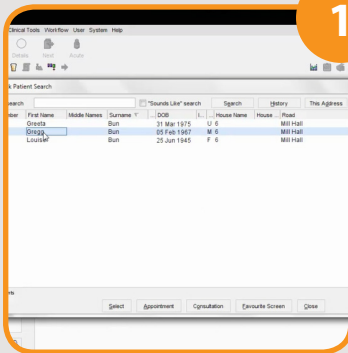

1 Choose a patient from Tpp SystemOne.
Minimise SystemOne from your desktop.




2 Open EasyOne Connect software. When it has opened you should see that the patient's details have been imported automatically. Fill in any required fields marked with an asterisk and click "OK".

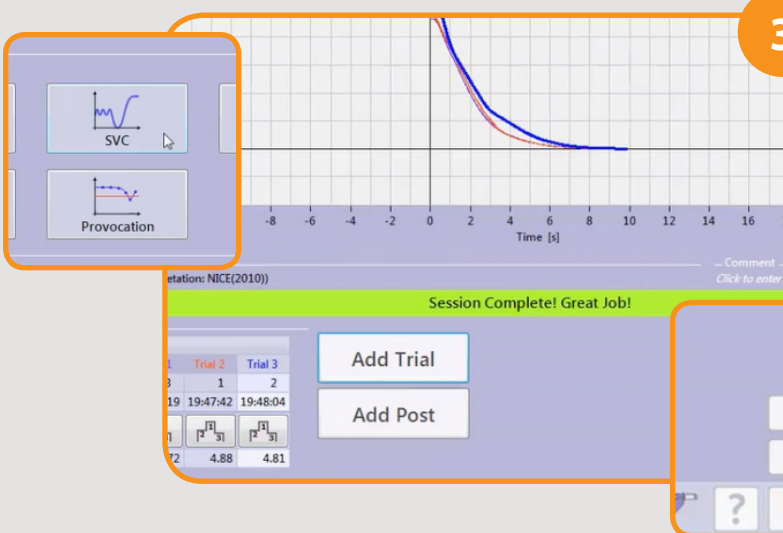


3 Choose SVC from the test screen and you will be asked to block the spirometer. This will set the spirometers baseline. Once this is done perform the SVC test.
After each test you can add a trial to perform another test.
Once you have done as many as required click on 'FINISH EMR'.

You will then get a box appearing saying attachment sent and parameters sent.

Then a box that says do you wish to perform another test. Click "YES".

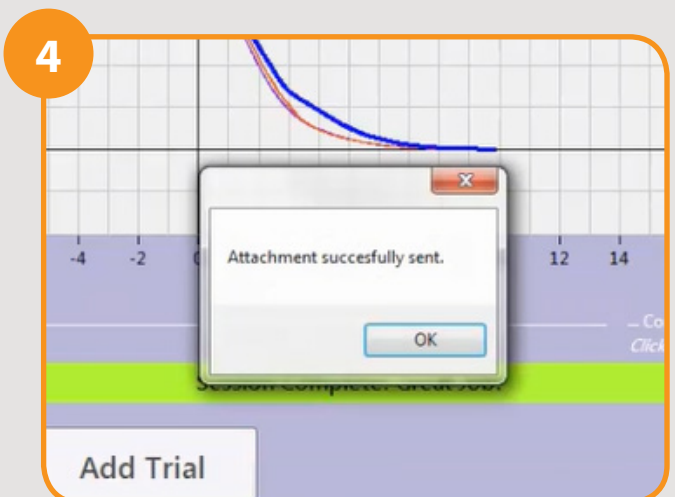

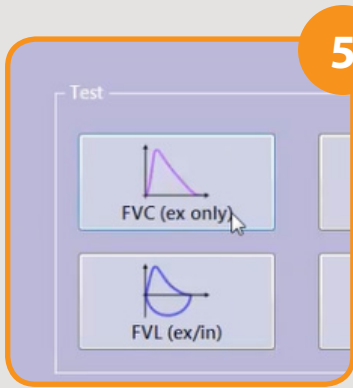

5



Confirm the patient details again and this time click on FVC exp only test.



6

Go through the test procedure again clicking add trial to perform more blows.

Once you have done as many as required click on 'FINISH EMR'.

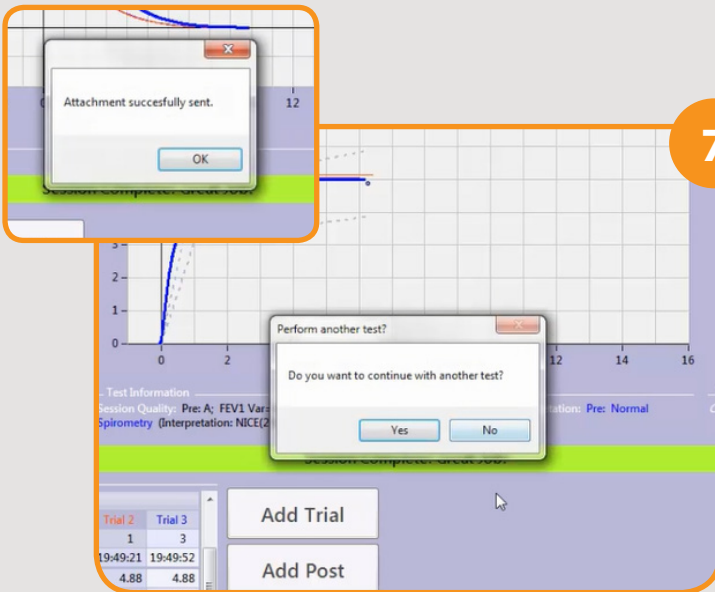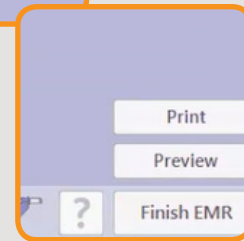

7

You will then get another box appearing saying "attachment sent" and "parameters sent".

Then a box that says "Do you wish to perform another test". Click "NO".

8

Exit the software and bring SystmOne back up to your desktop (remember you minimised it earlier).

Click on New journal and you will see all of the results and the attachment.