


How to upload data to WinSpiro PRO PC Software

This step by step guide has been designed to show users how to upload test result data from the Spirolab to the WinSpiro PRO PC Software.

1

Load the Winspiro PRO software.

Click on the "Data RC" icon which can be found on the top menu.

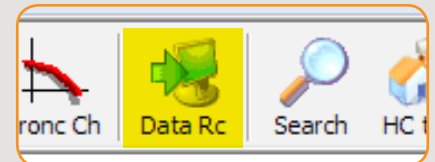


2

Connect the Spirolab Spirometer to the PC using the USB Cable provided.

3

Select "Receive via USB" from menu in the software

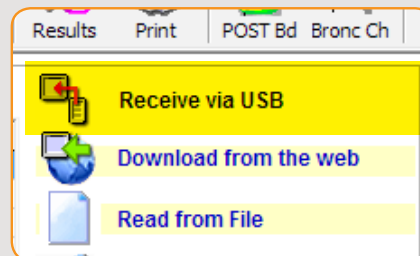

4

Once all of the data has been received from the Spirometer you should see a list of all tests downloaded.

Details of trials in selected file

Prog #	Gender	Age	cm	Kg	Date / Time	ID	Last Name	First Name	Link this Trial to the Patient...	Saved
8	Female	38	162	76	16/07/2013 12:51:17	8				No
7	Male	16			16/07/2013 14:44:58	1				No
6	Male	16			16/07/2013 14:14:23	1				No
5	Male	16			28/07/2015 11:17:16	1				No
4	Male	41	184	74	27/07/2015 17:10:21	3				No
2	Male	16			27/04/2015 13:52:02	1				No
1	Male	16			18/03/2015 14:54:15	1				No
1	Male	16			25/03/2015 23:46:58	1				No
1	Male	16			25/03/2015 23:45:26	1				No

Fix Data Area Input Data Area

Link/Unlink single Trial

The tests can be manually linked to an existing patient by clicking on any row in the "Link this trial to the Patient" column and choosing the patient from the drop down list

OR

Choose "Create Patients and Link" at the bottom of the screen to automatically create a patient in the database, then select "Save all linked Trials"

